



DTX Studio™ Clinic

Version 3.2

Instructions for Use

Table of Contents

Introduction	6
Disclaimer of Liability	6
Device Description	6
Intended Purpose	6
Intended Use / Indications for Use	6
Intended User and Intended Patient Target Group	6
Required Compatibility with Other Devices	6
Intraoral Sensors	6
Intraoral Cameras	7
Intraoral Scanning	7
Software	7
Devices with Measuring Function / Performance Characteristics	7
Contraindications	7
Cybersecurity	7
Compatibility	7
Interoperability	7
Intended Lifetime	7
Performance Requirements and Limitations	8
Clinical Benefits and Undesirable Side Effects	8
Facilities and Training	8
Notice Regarding Serious Incidents	8
Professional Use	8
System Requirements	8
Installation of the Software	8
Handling Instructions	8
Warnings, Cautions and Precautions	9
Warnings	9
Cautions / Precautions	10
System Requirements	11
Operating Systems	11
Devices	11
Start	12
Starting the Software	12
Exploring DTX Studio™ Home	12
Exploring the Notification Area	12

Adjusting Settings	13
Adjust the Default DTX Studio Home Settings	13
Import or Export Settings	13
Change Language and Date/Time Format	13
Adjust the DICOM Compliance Settings	13
Set the Default Image Filters	14
Disable Auto-Rotation of Intraoral Images	14
Disable Automatically Set Level and Window Values	14
Connect to DTX Studio Core	14
Add an Application to the Action Pane	15
Enable Practice Management System (PMS) Integration	15
Add a TWAIN-supported Device	15
Configure the Direct Folder for Detecting Third-Party Device Images	16
Viewing all Available Shortcuts	16
Closing the Software	16
Close the Software	16
Overview of the Main Features	17
Patient Records	18
Creating a New Patient Record	18
Managing Patient Records	18
Manage Privacy Options	18
Searching and Sorting Patient Records	19
Sort the Patients List	19
Search for a Patient	19
Exporting a Patient Record	19
Export a Patient Record	19
Manage Data	20
Importing Images from Third-Party Devices	20
Dragging and Dropping Images and Files into a Patient Record	20
Importing Data from Within DTX Studio Clinic	20
Import 3D X-rays	20
Import IO scans	21
Import Face Scans	21
Import 2D Images	21
Importing from 3Shape Dental Desktop Software	21
Sharing Patient Data via DTX Studio Go	22

Request Scans	23
Scheduling a Scan	23
Searching and Sorting Scan Requests	23
Sort the Scan Request List	23
Search for a Scan Request	24
Scan Workflows	24
Define a Scan Workflow	24
Apply a Scan Workflow	24
Perform a Scan	25
Perform a Scheduled Scan	25
Perform an Immediate Scan	25
Guided Acquisition with Intraoral Sensors or PSP Devices	25
Free Acquisition with Intraoral Sensors or PSP Devices	26
Guided Image Acquisition with Intraoral Cameras	27
Free Image Acquisition with Intraoral Cameras	27
Intraoral Scanning	27
3Shape TRIOS® Scanner	27
Dexis and Medit Scanners	28
Acquire Intraoral Scan Data	28
Reopen a Scan Case	28
Make a Diagnosis or Plan a Treatment	29
Exploring DTX Studio Clinic	29
Working with the Patient Menu	30
Menu Options	30
Diagnosis Tooth Chart	30
Edit the Tooth Chart	31
Diagnosis Data	31
Set the 3D and IO scan Viewer Background	32
Set the Default Image Zoom Level	32
Interact with the Mouse	32
Workspaces	33
Customize Workspaces	36
Show all Related Tooth Information with SmartFocus™	36
Customize Views with SmartLayout™	36
Align IO scans with SmartFusion™	36
Use the Thumbnail Bar	37
Add Diagnostic Findings	37
SmartPanel™ Tooth Chart	37
Adjust the Tooth Rotation Axis in the Tooth Workspace	38
Clipping the 3D Volume	38
Use Dental Intraoral Cameras in Workspaces	38
Tools	39

Reports	43
Create Reports	43
Add Custom Practice Logos	43
Opening DTX Studio Implant	43
Connect DTX Studio Clinic and DTX Studio Implant	43
Launch DTX Studio Implant	44
Orders	44
Order a Surgery Plan, Surgical Template or Restoration	44
Focus Area Detection	45
What is Focus Area Detection?	45
Using Focus Area Detection	45

Introduction

Disclaimer of Liability

This product is part of an overall concept and may only be used in conjunction with the associated original products according to the instructions and recommendations of Nobel Biocare, hereinafter referred to as 'the Company'. Non-recommended use of products made by third parties in conjunction with products of the Company will void any warranty or other obligation, express or implied. The user has the duty to determine whether or not any product is suitable for the particular patient and circumstances. The Company disclaims any liability, express or implied, and shall have no responsibility for any direct, indirect, punitive or other damages, arising out of or in connection with any errors in professional judgment or practice in the use of these products. The user is also obliged to study the latest developments in regard to this product and its applications regularly. In cases of doubt, the user has to contact the Company. Since the utilization of this product is under the control of the user, it is his/her responsibility. The Company does not assume any liability whatsoever for damage arising thereof. Please note that some products detailed in this Instructions for Use may not be regulatory cleared, released or licensed for sale in all markets.

Before using DTX Studio Clinic, please read these Instructions for Use and retain it for future reference. Please note that the information provided in this document is intended to get you up and running.

Device Description

DTX Studio Clinic is a software interface for dental/medical practitioners used to analyze 2D and 3D imaging data, in a timely fashion, for the treatment of dental, craniomaxillofacial and related conditions. DTX Studio Clinic displays and processes imaging data from different devices (i.e. intraoral X-Rays, (CB)CT scanners, intraoral scanners, intraoral and extraoral cameras).

DTX Studio Clinic features an AI-powered Focus Area Detection algorithm which analyzes intraoral radiographs for potential dental findings or capturing artifacts. The detected focus areas can be converted afterwards to findings after approval by the user.

Intended Purpose

Intended purpose of the software is to support the diagnostic process and treatment planning for dental and craniomaxillofacial procedures.

Intended Use / Indications for Use

DTX Studio Clinic is a software program for the acquisition, management, transfer and analysis of dental and craniomaxillofacial image information. It can be used to assist in the detection of the suspected dental findings and to provide design input for dental restorative solutions.

It displays and enhances digital images from various sources to support the diagnostic process and treatment planning. It stores and provides these images within the system or across computer systems at different locations.

Intended User and Intended Patient Target Group

DTX Studio Clinic is used by an interdisciplinary treatment team, to support them in treating patients who are subject to dental, craniomaxillofacial or related treatments.

Required Compatibility with Other Devices

DTX Studio ecosystem is compatible with the most used operating systems Windows and Mac including the latest releases.

Intraoral Sensors

DEXIS™ Titanium, DEXIS Platinum, GXS-700™, DEXIS IXS™, Gendex™ GXS-700.

Intraoral Cameras

DEXIS DexCAM™ 4 HD, DEXIS DexCAM 3, DEXIS DexCAM 4, Gendex GXC-300™, KaVo DIAGNOcam™ Vision Full HD, KaVo DIAGNOcam, CariVu™, KaVo ERGOcam™ One.

Intraoral Scanning

Compatibility with MEDIT Link Software* and MEDIT Scan for DTX Studio*, which supports the MEDIT™ i500/X 500 and MEDIT™ i700/X700 intraoral scanner or other compatible models.

Software

DTX Studio Core*, DTX Studio Implant, DTX Studio Go, DTX Studio Lab*, CyberMed OnDemand3D™*, Osteoid (formerly Anatomage) InVivo™.

* Product available for Windows operating system only.

Devices with Measuring Function / Performance Characteristics

The measurement accuracy and precision are 0.1 mm for linear measurements and 0.1 deg for angular measurements based on the input of (conebeam) CT-scans, acquired according to the instructions for use of the scanner equipment, with a voxel size of 0.5 mm x 0.5 mm x 0.5 mm.

DTX Studio Clinic reports the value, rounded to one digit after the decimal point, based on user-picked points.

Contraindications

N/A

Cybersecurity

It is recommended that active and up-to-date antivirus and anti-malware software, together with a correctly configured firewall, are installed on the computer where DTX Studio Clinic is to be used.

Furthermore, always lock the computer when it is left unattended. Failure to do so may lead to unintended handling of the diagnosis and planning or treatment.

Compatibility

DTX Studio Clinic is connected with other medical devices and compatible with previous DTX Studio Clinic versions.

Interoperability

DTX Studio Clinic is interoperable with:

- DTX Studio Core.
- DTX Studio Implant.
- DTX Studio Go.
- DTX Studio Lab.
- CyberMed OnDemand3D.
- MEDIT Scan for DTX Studio.
- Medit Link.

Intended Lifetime

For software the intended lifetime is three years. When used on the supported operating systems, the software will keep performing according to its intended use.

Performance Requirements and Limitations

It is important to make sure DTX Studio Clinic is used only with approved operating systems. See the [System Requirements](#) in the IFU for more information.

Clinical Benefits and Undesirable Side Effects

DTX Studio Clinic is a component of dental or craniomaxillofacial treatment. Clinicians can expect the software to support the diagnostic and treatment planning process.

No undesirable side effects identified for DTX Studio Clinic.

Facilities and Training

It is strongly recommended that clinicians, new as well as experienced users of implants, prosthetics and associated software, always go through special training before undertaking a new treatment method.

Nobel Biocare offers a wide range of courses for various levels of knowledge and experience.

For more information please visit our training website on tw.dtxstudio.com.

Notice Regarding Serious Incidents

If, during the use of this device or as a result of its use, a serious incident has occurred, please report it to the manufacturer and to your national authority. The contact information for the manufacturer of this device to report a serious incident is as follows:

Nobel Biocare AB

<https://www.nobelbiocare.com/complaint-form>

Professional Use

DTX Studio Clinic is for professional use only.

System Requirements

We advise to check the [System Requirements](#) before starting the installation of the software. To obtain information on minimum and/or recommended requirements, please contact customer support. New versions of the software may require higher requirements for hardware or operating system.

Installation of the Software

To obtain information on how to install the software, please contact the authorized technician or customer support.

Handling Instructions

For detailed information on how to use the software, please refer to the detailed instructions further on in this IFU.

Warnings, Cautions and Precautions

Warnings

Following warnings are shown in the software.



- **The name in the DICOM file(s) differs from the patient name.**

To reduce the risk of using incorrect data to create the patient model, verify the patient name and check whether patient name and the name in the used DICOM set correspond.
- **Cannot add the 3D scan to the current diagnosis.**

The current diagnosis contains a 3D scan linked to a surgery plan. Create a new diagnosis to import the 3D scan.
- **Cannot add the surgery plan to the current diagnosis.**

Select a surgery plan based on the 3D scan included in the current diagnosis.
- **Exporting images in 8 bit could potentially result in fidelity loss.**

It is recommended to export images in another format to maintain their quality.
- **Do not expose patient.**

Device could not be armed. In this state, the device cannot receive X-ray radiation. Retry by reconnecting or restarting the device. If the problem is not resolved, please contact the customer support of your device.
- **Preparing your sensor for the next exposure. Please wait.**

Device is currently re-arming. In this state, the device cannot receive X-ray radiation.
- **Verify the scan request parameters on the device.**

Before exposing the patient, make sure to check the parameters on the device.
- **It is not advised to modify the treatment plan without the use of the actual implant shapes.**

The actual shapes can be downloaded from DTX Studio Go.
- **Image has been flipped.**

This warning is displayed when images are manually flipped (horizontally or vertically) by users.
- **Automatic intraoral images sorting (MagicSort™) is intended to be used only for adult dentition without gemination, crowding and macrodontia.**

To reduce the risk of using MagicSort on non-suitable patient images.

Furthermore, a number of technical warnings (e.g., inconsistent CT data) are visualized in DTX Studio Clinic.

It is strongly recommended that users follow the instructions and technical notifications within the software to reduce the risk of an inaccurate scan.

Automatic intraoral images sorting (MagicSort) is intended to be used only for adult dentition without gemination, crowding and macrodontia.

The clinician should not rely solely on the output identified by focus area detection, but should perform a full systematic review and interpretation of the entire patient dataset and other differential diagnostic methods.

Focus area detection is limited to images where detection can be performed.

The automatic detection of focus areas is intended to be used only for adult dentition without gemination, crowding and macrodontia.

Cautions / Precautions



- Users are advised to obtain training before undertaking a new treatment method or using a new device.
- When using a new device or treatment method for the first time, working with a colleague who is experienced with the new device or treatment method may help avoid possible complications.
- The user should ensure patient movement is kept to a minimum during the scanning process to reduce the risk of an incorrect scan.
- The lack of knowledge and comprehension of the software may result in the delay in or rescheduling of the diagnosis and planning or the actual treatment.
- When using the diagnostic and planning tools provided in the software, it is important to pay extra attention to:
 - the correctness of made indications (visualizations, measurements, critical structures, imported data, implant planning).
 - the correctness of the outcome of the automated functions (the alignment of dental scans, the autofilling of holes, and airway segmentation)
 - the correctness of the patient ID (after opening a patient record through PMS systems and when creating scan requests).
 - the data being up to date and not outdated.

Failure to do so increases the risk of the need for revision of the diagnosis and planning or treatment which, in its turn, may lead to the delay in or rescheduling of the diagnosis and planning or the actual treatment.

- It is recommended to take extra care when working with image capturing devices. Incorrect use may result in the delay in or rescheduling of a diagnosis and planning or treatment or in unnecessary extra radiation exposure of the patient.
- When extracting a report or patient data from the software, it is important to know that not de-identified patient data can be used for incorrect purposes without the patient's consent.
- It is recommended to pay extra attention to the assigned tooth numbering and orientation markings of the viewers. A wrongly assigned tooth number or incorrect patient orientation may lead to incorrect treatment actions performed on the patient.
- After updating the software version, it is recommended to verify the critical settings of the open patient cases and/or treatment plan to make sure these settings are correct in the new software version. Incorrect settings may lead to a delay in or the rescheduling of the diagnosis and planning or the actual treatment.

System Requirements

Operating Systems

- Windows® 64-bit (Pro and Enterprise edition):
 - Windows 10
 - Windows 11
- Intel-based Mac (*):
 - macOS Big Sur (11)
 - macOS Monterey (12)
 - macOS Ventura (13)
- Apple Silicon Mac (M1 Chip or Higher):
 - macOS Big Sur (11)
 - macOS Monterey (12)
 - macOS Ventura (13)

(*) All Intel-based models since 2012 are supported.

Devices



- Windows desktop
- Windows notebook
- iMac®, Mac® Mini, Mac Pro®, MacBook Pro®, MacBook Air® (*)

* The graphics cards of some MacBook Air® and Mac® Mini configurations have restrictions with regard to volume rendering. Consider selecting low resolution volume rendering.

	Basic setup (2D imaging only)	Recommended setup (2D and 3D imaging with better performance)
CPU	Dual or quad-core	2.8 GHz quad-core (Intel Core i5 or i7)
RAM	4 GB	8 GB or more
Graphics card	Entry-level dedicated add-in card or Intel integrated graphics. 6th generation Intel® CPUs or higher are supported (with embedded 9th generation GPU or higher). OpenGL® 3.3 support is required**.	Dedicated add-in graphics card with optimal 3D support (OpenGL 3.3) and 2 GB memory or more (such as AMD or NVIDIA). For 4K displays: 4 GB memory or more. 6th generation Intel® CPUs or higher are supported (with embedded 9th generation GPU or higher).
	* Always use the latest integrated graphics card driver available from Intel for the respective model for best performance. ** To check the OpenGL® version of your graphics card, go to http://realtech-vr.com/admin/glview .	
Disk space	10 GB free disk space for installation and additional disk space for user-created data. A typical 2D patient dataset in DTX Studio Clinic is about 10 MB.	10 GB free disk space for installation and additional disk space for user-created data. A typical 3D patient dataset in DTX Studio Clinic is about 250 MB.
Network	Broadband Internet connection with 3Mbps upload and 30 Mbps download speed. It is recommended always to be connected to the Internet. If that is not possible, a connection should be established at least once every 14 days, because otherwise your access to DTX Studio Clinic may be temporarily suspended.	
Hard Disk	Only install DTX Studio Clinic onto an HFS+ or HFSJ non-case-sensitive drive on Mac devices.	
Monitor	Full HD (1920x1080) or higher. Information may appear missing if display scaling is used. For this reason, the equivalent scaled resolution should not be lower than 1920x1080.	
LAN	If DTX Studio Clinic is installed together with DTX Studio Core a local Gigabit Network is recommended.	

Start

Starting the Software

1. Open DTX Studio Clinic:
 - On Windows, double-click the shortcut icon  on the desktop.
 - On macOS, click the shortcut icon  in the Finder application folder or the Dock.
2. Select the user.
3. Enter your password.
4. Click [Log in](#).


Notes

If two-factor authentication was set up on DTX Studio Go, every 30 days you will need to enter a six-digit verification code to log in.

DTX Studio Clinic should always be connected to the Internet. If that is not possible, a connection should be established at least once every 14 days, because otherwise your access to DTX Studio Clinic may be temporarily suspended.



Exploring DTX Studio™ Home

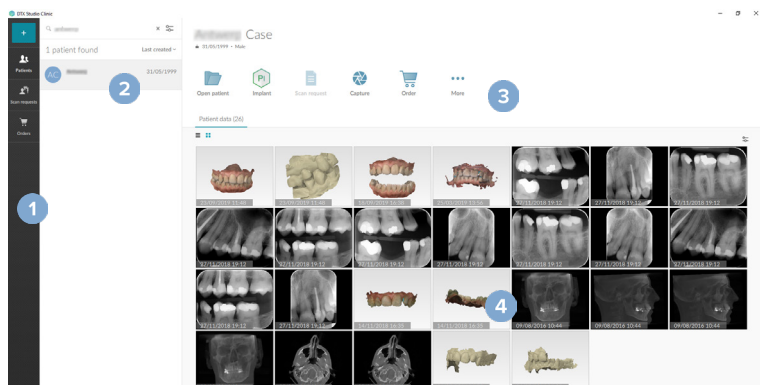






DTX Studio Home is the work area where you select and manage patient records, scan requests, orders and general settings.



- 1 Sidebar
- 2 Patient list
- 3 Action pane
- 4 Details pane

Exploring the Notification Area



The  icon in the notification area grants access to the DTX Studio Home settings () and the following tabs:

- **Notifications:** shows which patient records are being uploaded or synchronized with DTX Studio Core.
- **DTX:** provides quick access to DTX Studio Core () , DTX Studio Go () , Exposure reports*, QuickPrescribe*, Scan Center* or a linked third-party application.
- **Devices:** lists direct folders for importing third-party device images, shows acquisition devices and their statuses (connected via USB or TWAIN  , online  , busy  or offline ). Click  to access the device settings, folder import settings or to disable unrequired devices.

*Requires DTX Studio Core.

Adjusting Settings


Adjust the Default DTX Studio Home Settings

1. Click **Menu** .
2. Click **Settings** .

Import or Export Settings

Create or import a settings file containing set preferences in DTX Studio Home or in DTX Studio Clinic. This can be useful for exporting settings to a new installation, sharing settings with other DTX Studio Clinic users, or making a backup.

Note

Consult the “Share settings” topic in the help files for an extensive overview: Click  and select **Help**.

In the DTX Studio Home [settings](#) sidebar, click **Share settings**.

- To import a settings file, click **Browse** in the **Import** section. Select the settings file and click **Save**. Click **Import** to confirm.
- To export a settings file, click **Browse** in the **Export** section. Change the filename if needed and click **Save**. Click **Export** to confirm.

Change Language and Date/Time Format

To adjust the preferred language, date format and time format:

1. In the DTX Studio Home [settings](#) sidebar, click **Regional**.
2. Select the date and time format in the **Short date format**, **Long date format** and **Time format** lists.
3. Select the preferred language from the **Application language** list.
4. Click **OK**.
5. Restart DTX Studio Clinic for the changes to take effect.

Adjust the DICOM Compliance Settings

To ensure compliance with the DIN 6862-2 standard, enter the institution information. When exporting a DICOM file, the provided institution information replaces empty tags.

1. In the DTX Studio Home [settings](#) sidebar, click **Practice**.
2. Select **Use DIN 6862-2 standard**.
3. Provide the requested institution information.
4. Click **OK**.

Note

When importing and exporting a compliant DICOM file, the DIN 2020 tags always remain preserved.

Set the Default Image Filters

To set the default image filters for the DTX Studio Home viewer and DTX Studio Clinic:

1. In the DTX Studio Home [settings](#) or DTX Studio Clinic preferences sidebar, click [Image settings](#).
2. From the [Default image settings](#) list, select the image type for which you want to set the default filter settings.

Note

To set custom values for the Gamma filter, select [OPG](#), [Intraoral](#) or [Ceph](#) in the [Default image settings](#) list. Set the [Gamma](#) option to [Manual](#).

3. Select the filters to be used by default for the selected image type and use the appearing slider to set the filter percentage.
4. Click [OK](#).

If you wish to revert to the initial default values, click [Reset](#).

Disable Auto-Rotation of Intraoral Images

Upon performing an immediate scan, intraoral images are automatically rotated in the correct position. To disable this:

1. In the DTX Studio Home [settings](#) or DTX Studio Clinic preferences sidebar, click [Image settings](#).
2. Deselect [Apply auto-rotation in DTX Studio capture](#).

Disable Automatically Set Level and Window Values

Upon importing or acquiring a 2D image, level and window values are automatically set. To disable this:

1. In the DTX Studio Home [settings](#) or DTX Studio Clinic preferences sidebar, click [Image settings](#).
2. In the top-right drop-down menu, select [OPG](#), [Intraoral](#) or [Ceph](#).
3. Deselect [Auto level window](#).
4. Enter custom level and window values.
5. Click [OK](#).

Connect to DTX Studio Core

DTX Studio Core is a software solution to store and retrieve patient media and image data types (2D X-ray, 3D (CB)CT X-ray, dental optical scans, photos) in a structured and centralized way, so that stored data is immediately accessible anywhere in the dental clinic.

- If connected with DTX Studio Core, DTX Studio Clinic can be used in a network environment to acquire images from other supported Ethernet-connected and 3Shape TRIOS® devices.
- A connection with DTX Studio Core needs to be established to work with network devices, scan requests and access radiographic reports.

Start

To establish a connection with DTX Studio Core:

1. In the DTX Studio Home [settings](#) sidebar, click **Core**.
2. Enter the **URL** (web address), **Login** and **Password** for DTX Studio Core.
3. Click **Connect**.

Add an Application to the Action Pane

To add an application shortcut to the action pane:

1. In the DTX Studio Home [settings](#) sidebar, click **Quick Launch**.
2. Click **Add**.
3. Select the executable file and click **Open**.
4. If needed, change the **Application name**.
5. Optionally, select **Launch with patient data** to launch the third-party application with patient data.
 - Specify which data is exported by adding export parameters to the **Export parameters** field.

Note

Consult the Quick launch topic in the help files for an extensive overview of all patient data parameters.

- Click **Browse** to choose a location for the exported data.

6. Click **OK**.


Enable Practice Management System (PMS) Integration

Integrating DTX Studio Home with a PMS (i.e. through VDDS or OPP) /OPP web allows you to create a patient record and acquire an image from within the PMS.

Preview the PMS images in DTX Studio Home or view them directly in DTX Studio Clinic.

1. In the DTX Studio Home [settings](#) sidebar, click **PMS integration**.
2. Select **Enable PMS integration**.

Note

Consult the “PMS integration” topic in the help files for detailed information: Click  and select **Help**.

Add a TWAIN-supported Device

1. In the DTX Studio Home [settings](#) sidebar, click **Devices**.
2. Click **Add**.
3. Select the TWAIN device.
4. Configure the device settings.
5. Click **Add**.

Configure the Direct Folder for Detecting Third-Party Device Images

To add images from third-party camera devices or third-party (CB)CT devices, set up a direct folder in which new images will be detected. Add them with the [Capture](#) action in a patient record or from within DTX Studio Clinic.

1. Prepare your camera device settings:
 - If possible, set up your third-party camera device, (CB)CT device or wireless SD card to store images in a specific folder.
 - If the images are stored on a standard SD card, insert it and take note of the assigned drive letter.
2. In the DTX Studio Home [settings](#) sidebar, click [Devices](#).
3. Click [Add](#).
4. Select the third-party device and click [Select](#).
5. Click [Browse](#) to select the camera device folder and click [Select folder](#).
6. Enter a specific name.
7. Change the modality and folder priorities, if needed.
8. Click [Add](#).

Viewing all Available Shortcuts

To view all available shortcuts, click [Menu](#) ☰ and select [Keyboard shortcuts](#).


Closing the Software

Make sure to close all active DTX Studio Clinic and scan module* instances.

Close the Software

1. Click [Menu](#).
2. Click [Close application](#).








Note

Using the closing button , the software remains running in the background, to allow data synchronization and a quicker response when reopening DTX Studio Home / DTX Studio Clinic.







*Module might be licensed.

Overview of the Main Features

To get started with the main features:



-
- | | | |
|---|---------------------------------|--|
| 1 | Create or link a patient record |  Create a patient record in DTX Studio Home (see page 18). |
| | |  Integrate your Practice Management System (see page 15) and link an existing PMS patient record. |
-
- | | | |
|---|------------------------|--|
| 2 | Acquire or import data |  Acquire scans, initiate multiple scan workflows (see page 24) or import third-party device images from direct folders (see page 16). |
| | |  Request a scan or multiple scan workflows (see page 24). |
| | |  Drag and drop images in a patient record (see page 18). |
| | |  Migrate data on demand (for setups with database migration) (see page 32). |
| | |  Import from 3Shape Dental Desktop Software (see page 21). |
-

Once a patient record is created and data are added, continue to:

-
- | | |
|------------------------------|--|
| Diagnose and plan treatments |  Open DTX Studio Clinic (see page 29). |
| |  Optionally, open DTX Studio Implant (see page 43). |
-
- | | |
|-----------------------|--|
| Share and communicate |  Share data via DTX Studio Go (see page 22). |
| |  Export a patient record (see page 19). |
| |  Create a report or patient communication letter (see page 43). |
-
- | | |
|--------------|---|
| Place orders |  Optionally, order a restoration, surgical template or surgery plan (see page 44). |
|--------------|---|
-

Patient Records

Creating a New Patient Record




1. Click **Add patient** .
2. Select **Create patient**.
3. Enter basic patient details, like patient name, date of birth and gender.
4. Click **Create**.
5. The patient record is added to the **Patients**  list. If DTX Studio Home is connected to DTX Studio Core, the patient record is also added to DTX Studio Core.

Managing Patient Records

Click **Patients**  in the sidebar to open the patients list, if not already open.

Note


If the workstation is not connected to DTX Studio Core, only locally-stored patient records are shown.

- Patient records opened in DTX Studio Clinic on a local or network-connected workstation are marked with .
- To edit basic patient information, select the patient record in the patients list, click **More ...** and select **Edit** .
- To delete a patient, select the patient record in the patients list, click **More ...** and select **Delete** .
- To make sure that a patient record stored in DTX Studio Core is also available offline, click **More ...** and enable **Offline available**.
- Add compatible file types or images to the patient record by dragging and dropping them on the patient data field. Indicate the type. Click **Import**.


The notification area ([see page 12](#)) shows which patient records are being uploaded or synchronized.


Manage Privacy Options

To ensure patient privacy by only showing patient initials in the patients list or completely hiding the patients list:

1. On the patients list, click .
2. Select **Privacy mode** to show initials only or **Hide patients list** to completely hide it.

Notes

Click  in the sidebar to re-show the patients list.

Privacy mode remains enabled, even when restarting DTX Studio Clinic. To turn off the privacy mode, re-click  and uncheck **Privacy mode**.



Searching and Sorting Patient Records

To find a patient record, sort the patients list or use the search function.

Sort the Patients List

1. Click the drop-down arrow next to the patients list header.
2. Select [Last created](#), [Last modified](#) or [Last captured](#).
3. Click the drop-down list again to close it.

Search for a Patient

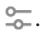1. On the patients list, click .
2. Select to search for the [Patient name](#), [Date of birth](#) or [Patient ID](#).
3. Type (a part of) the selected search option in the [Find a patient](#) field .
4. While typing text in the search box, the patients list is filtered automatically.

To remove the search criteria, click **x** in the search field.


Exporting a Patient Record

Export a patient record to share the diagnoses and image data with another DTX Studio Clinic user. Additionally, the exported patient can be viewed in the free version of DTX Studio Clinic, available via DTX Studio Go.

Note

When extracting a report or patient data from the software, it is important to know that not de-identified patient data may be used for incorrect purposes without the patient's consent.


Export a Patient Record

1. Select the patient record in the patients list.
2. Click [More ...](#).
3. Click [Export patient](#) .
4. Select the data you want to export.
5. Browse to the location where to store the data and click [Export](#).

Manage Data

Importing Images from Third-Party Devices

To add images from third-party camera devices or third-party (CB)CT devices, make sure to configure a direct folder in which new images will be detected ([see page 16](#)).

1. Select the patient record in the patients list or open a patient record in DTX Studio Clinic.
2. Click **Capture** .
3. Hover over a direct folder name and click **Select**.
 - For 2D images, select the images you want to import. Click **Finish**.
 - For 3D (CB)CT devices, select the 3D data you want to import. Click **Import**.


Dragging and Dropping Images and Files into a Patient Record

1. Drag and drop a compatible image or file type from the file explorer into a patient record.
2. If needed, change the modality and the acquisition date.
3. Click **Import**.
4. The image or file is added to the patient record.

Importing Data from Within DTX Studio Clinic

To import data, click **Import**  in the patient menu.

Import 3D X-rays

1. Click **Import**  and select **3D X-ray**.
2. Select the DICOM data.
3. Click **Next**.
4. Set the isovalue.
5. Click **Finish**.
6. If MagicDetect is enabled (on by default), the AI detection process is started to automatically set up the (CB)CT data.
 - Click **Skip magic detection** to manually indicate the landmarks and OPG curve later.
 - The 3D image orientation can be optimized via the patient orientation wizard.
 - The OPG curve action ([see page 39](#)) allows you to adjust the 3D OPG.

Import IO scans

1. Click **Import** ↓ and select **IO scan**
2. Select the IO scan model(s) and add acquisition details, if needed.
3. Click **Next**.
4. Click **Finish**.

Import Face Scans

1. Click **Import** ↓ and select **Face scan**.
2. Select a face scan to import.
3. Click **Open**.
4. If needed, adjust the **Brightness** and **Contrast**.
5. Click **Done**.

Import 2D Images

1. Click **Import** ↓ and select **2D image**.
2. Choose the image(s) and click **Open**.
3. Click **OK**.
4. Select the images you want to add.
5. Click **Import**.
 - Upon importing or acquiring a 2D image, level and window values are automatically set. To disable this, [see page 14](#).
 - By default, MagicSort autodetection is activated. This can be deactivated in the settings.

Importing from 3Shape Dental Desktop Software

To import a scan that was acquired with a 3Shape TRIOS intraoral scanner, a patient record needs to be created in DTX Studio Home first.

Note




For more information on how to integrate a 3Shape TRIOS intraoral scanner device, see the DTX Studio Core Quick Guide.

1. Create a new patient record in DTX Studio Home. To make sure that the 3Shape data will merge:
 - Use exactly the same first name, last name and date of birth as those of an existing patient in 3Shape Dental Desktop.
 - Make sure that the patient names are correctly capitalized. Names are case-sensitive.

Notes





Previously imported patient records cannot be re-imported. Duplicate the patient record in 3Shape Dental Desktop, and import the duplicated patient record instead.

The DTX Studio Clinic patient record ID is not exchanged with 3Shape. The link is created based on the user confirmation that the first name, last name and date of birth are identical.

2. Select the patient record in the **Patients**  list.
3. Click **More** .
4. Select **Import from 3Shape** .
5. If requested, confirm that the patient record matches the one in the 3Shape Dental Desktop.
6. The data is retrieved and added to the **Patient data** tab on the patient detail panel.

Sharing Patient Data via DTX Studio Go

Share patient data with a dental office via [DTX Studio Go](#). If patient data is shared, an additional **GoShare** tab shows an overview.


1. On a patient record's action pane, click **Share** .
2. Select what you want to share:
 - **Full patient**  to share the entire patient record (DTX Studio Clinic proprietary encrypted format), or
 - **Patient data**  to share specific data.
3. Choose the preferred options.
4. Click **Continue**.
5. DTX Studio Go opens in the web browser and the created GoShare case is shown. Meanwhile, the data is uploaded in the background.
 - Add notes to the **Prescription** card's text field.
 - To create a prescription note template, click **Insert quick note** and select **Configure**. Click **Add quick note**. Add a title, write the custom text and click **Save**. Click **Close**.
 - To change the default prescription image or to add annotations, hover over the prescription image and select **Edit prescription**. Click **Change image** to select another image.
 - To edit the patient information, click  in the top-right corner.
 - If needed, provide further information or additional files of the (CB) CT images, clinical pictures, intraoral images, OPG, reports etc.
6. Click **Start sharing**.
7. Select the connection with which you want to share patient data. Either by searching or selecting an existing connection in the **Share with a connection** field, or by entering an email address.
8. Click **Send**. The receiving account is notified by e-mail.
9. The shared case is added to the **GoShare** overview in the patient record. Click **View case** to open the shared case in DTX Studio Go.

Request Scans

To work with scan requests or to initiate a scan workflow with multiple scan protocols ([see page 24](#)), a connection with DTX Studio Core needs to be established ([see page 14](#)).

Scheduling a Scan

To request a scan for a patient:

1. Select the patient record in the patients list.
2. Click [Scan request](#) .
3. Optionally, initiate a scan workflow with multiple scan protocols ([see page 24](#)).
4. Hover over the picture tile of the image acquisition device and click [Select](#).
5. Fill out the applicable information in the scan request form.


Note

Depending on the chosen modality, the form is different.


- If the image acquisition device allows multiple modalities, select the required modalities: [3D](#), [CEPH](#) (Cephalogram), [OPG](#) (PAN), [IOXRAY](#) (2D intraoral scan), [IOS](#) (3D intraoral scan) and/or [IOCAM](#) (intraoral photo). If applicable, select an imaging program.
 - On the tooth chart, select the regions you want to scan.
 - If needed, change the [Scan date](#) and the [Requesting clinician](#).
 - Select [Sinus](#) if the sinus needs to be scanned.
 - Select an image acquisition [resolution](#) if applicable.
 - If the selected device is an intraoral device, select a [Template](#), and specify the [Imaging program](#).
 - Add [Request notes](#) for the operator if applicable.
6. Click [Create scan request\(s\)](#). The scan request is added.

Searching and Sorting Scan Requests

Sort the Scan Request List

1. In the sidebar, click [Scan requests](#) .
2. Click the drop-down arrow next to the [Scan requests](#) list header.
3. Select to sort by [Scheduled date](#) or [Creation date](#).

Search for a Scan Request

1. Type (a part of) the scheduled date or the patient name in the [Find a scan request](#) field .
2. While typing text in the search box, the scan requests list is filtered automatically. The search results are sorted on the scheduled date.


To remove the search criteria, click **X** in the search field.

Scan Workflows

Use the QuickPrescribe feature to create a scan workflow consisting of multiple scans from different modalities with a particular set of pre-defined scan requests. These scan workflows can then be used on all workstations.

Define a scan workflow in DTX Studio Core first, and apply it during the schedule a scan or capture wizards.

Define a Scan Workflow

1. Click the  icon in the notification area at the bottom of the screen (on the top of the screen if you use a Mac).
2. Click [QuickPrescribe](#). If you have not set up any scan workflows yet, click [Get started right away](#).
3. Complete the scan workflow configuration in DTX Studio Core.



Note

In DTX Studio Core, click [Help](#) in the bottom-left corner for more information.

4. Select a scan workflow from either the scan request or capture wizard.

Apply a Scan Workflow

Define a scan workflow in DTX Studio Core, and select it during the scan request wizard or capture wizard.

1. Select the patient record in the patient list.
2. Click [Scan request](#)  or [Capture](#) .
3. Click [QuickPrescribe](#).

Notes

Type (a part of) the scan workflow in the search field to narrow down the results.

To see all the modalities and parameters that are set, hover over a scan workflow name in the list. Click [More info](#).

4. Hover over a scan workflow name in the list, and click [Create scan requests\(#\)](#). The number indicates the number of scan requests created with the selected scan workflow.
5. On the tooth chart, select the diagnostic region that you want to scan.
6. Click [Create scan requests](#).

Perform a Scan

Take a scan before, during or after diagnosing the patient — with or without creating a scan request first.


It is strongly recommended that users should follow the instructions and technical notifications within the software to reduce the risk of an inaccurate scan.

Perform a Scheduled Scan

To perform a scan for a scan request:

1. On the scan request card, click **Start**.
2. Hover over a device tile and click **Select**.
3. Go to the device to perform the scan.
4. Click **Finish** to end the action, or click **Open diagnose** to open the patient record in DTX Studio Clinic.

Perform an Immediate Scan

1. Select the patient record in DTX Studio Home or open a patient record in DTX Studio Clinic.
2. Click **Capture** .
3. Hover over a device or direct folder tile and click **Select**.

Notes

Intraoral images are automatically rotated in the correct position. To disable this, [see page 14](#).

Upon importing or acquiring a 2D image, level and window values are automatically set. To disable this, [see page 14](#).

Use multiple sensors with different sensor sizes by plugging them in or out during the capture wizard. The used sensor is shown in the top-right corner*.

If multiple sensors are plugged in, the + symbol appears*. All sensors which are connected and active, are ready to acquire the scan. X-ray triggers the image acquisition.

* For sensors and PSPs directly supported in DTX Studio Clinic. For devices connecting via TWAIN, this functionality will be limited.

Guided Acquisition with Intraoral Sensors or PSP Devices

To acquire intraoral images with a template:

1. Initiate an immediate scan.
2. Click the **Template** tab and select your preferred template.
3. Select the layout and images that you want to acquire.
4. Click **Start**.
5. Go to the device to perform the scan.
6. On the preview page, inspect the acquired images. Hover over a preview image to see the additional **Rotate**, **Flip**, show or hide **Image filters** and **Capture again** options. Make changes if needed.
7. Click **Finish**.

Free Acquisition with Intraoral Sensors or PSP Devices

To acquire intraoral images without using a template:

1. Initiate an immediate scan.
2. Select the **Free** tab.
3. Go to the device to perform the scan.



If you have activated MagicSort autodetection, see the steps below:

- In the bottom-right corner, the MagicSort autodetection icon is shown.
- In the capture wizard teeth are automatically detected. They are marked in blue. Click on a tooth to remove the MagicSort label.
- Intraoral images are automatically mapped on FMX charts.
- If needed, manually indicate unidentified images on the tooth chart.

Note

By default, MagicSort autodetection is activated. This can be deactivated in the settings.

4. On the preview page, inspect the acquired image and assign a tooth range if necessary.
 - Click **Clear selection** to remove the indicated teeth on the tooth range.
 - Make changes if needed: hover over a preview image to see the additional **Rotate**, **Flip**, show or hide **Image filters** and **Capture again** options.
 - Rotate or flip the acquired image if necessary.

Action	Icon	Shortcut
Rotate image counterclockwise		Alt + , or R
Rotate image clockwise		Alt + , or Shift+R
Flip intraoral image or clinical picture horizontally		U
Flip intraoral image or clinical picture vertically		Shift+U

Guided Image Acquisition with Intraoral Cameras

To acquire intraoral camera images with a template:

1. Initiate an immediate scan.
2. Stay on the **Guided** tab.
3. For intraoral cameras, select the teeth that you want to acquire.
4. Press the device button if available or click **Capture image**.
5. To select another tooth of which intraoral images will be taken, use the **←** or **→** keyboard buttons. Alternatively, click the tooth on the tooth range or click **Previous** or **Next**.

Note

When only one image per tooth is required, enable **Proceed to the next tooth after capture** to automatically continue with the next tooth.

6. Click **Finish**.

Free Image Acquisition with Intraoral Cameras

To acquire intraoral camera images without using a template:

1. Initiate an immediate scan.
2. Click the **Free** tab.
3. Press the device button if available or click **Capture image**.
4. To assign acquired images to a tooth, click an image thumbnail at the bottom and select the corresponding tooth on the tooth range.


Note

Assign an image to multiple teeth by selecting the image, clicking a tooth and dragging over the other teeth.

5. Click **Finish**.

Intraoral Scanning

3Shape TRIOS® Scanner

1. Click **Capture** .
2. Select 3Shape intraoral scanner and click **Select**.
3. Start the scan in the 3Shape Dental Desktop application.

Note


For more information on how to integrate a 3Shape TRIOS intraoral scanner device, see the DTX Studio Core Quick Guide.

4. Complete the scanning process.
5. The image is added to the **Patient data** tab on the patient detail panel.
 - The finished scan request is marked with a checkmark.
 - Click **Open diagnose** to open the patient record in DTX Studio Clinic.

Dexis and Medit Scanners

The scan module* allows you to use a supported Medit / Dexis intraoral scanner with DTX Studio Clinic and to reopen existing scan cases.

Acquire Intraoral Scan Data

1. Select the patient record in the patients list.
2. Click **Capture** .
3. Select the intraoral scanner and click **Select**.
4. The scan module is opened.
5. Follow the instructions.
6. The processed data is added to the patient record.

Reopen a Scan Case

Intraoral scan data that was locally captured can be reopened in the scan module*.

Note

This is only possible on the computer used to acquire the scan data.


To reopen the scan module* to edit the scan, take additional scans, trim, measure and more:

1. In a patient record, click the IO scan tile.
2. Click **Open in Medit Scan**.

*For Windows computers only and on the computer which has been used for acquiring the intraoral scan data, where the folder with RAW scan data is available. An appropriate license type or a subscription to the Plus+ feature Medit Scan integration is required. The additional Medit Scan software module has to be installed.

Make a Diagnosis or Plan a Treatment


Use DTX Studio Clinic to view and inspect the patient data and to add findings and measurements to the patient record.

In DTX Studio Home, select a patient in the **Patients** list and click **Open Patient** . Alternatively, double-click the patient in the patient list or press [O].

The features depend on the DTX Studio Clinic license type:

License Type	Features	Image Acquisition
DTX Studio Clinic Pro or Pro IOS	2D and 3D	2D and 3D
DTX Studio Clinic Select	2D and selected 3D viewing features	2D only

Exploring DTX Studio Clinic

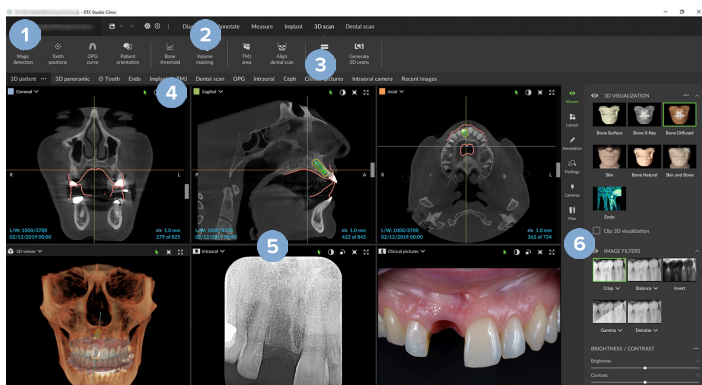

- 1 Patient menu
- 2 Menu bar
- 3 Toolbar
- 4 Workspace bar
- 5 Workspace - SmartLayout
- 6 SmartPanel

Working with the Patient Menu


To open the patient menu in DTX Studio Clinic, click ☰ in the upper-left corner.

Menu Options

- **Capture:** acquire data directly.
- **Import:** import data to the open diagnosis.
- **New:** create a new diagnosis.
- **Open:** open another existing diagnosis.
- **Save:** save the open diagnosis.
- **Export report:** export a report of the patient diagnosis.
- **Keyboard shortcuts:** show the keyboard shortcuts overview.
- **Preferences:** change the settings, like the **Default workspace**, **Performance**, **Image settings**, **3D viewer** and **Tools**.
- **Close patient:** close DTX Studio Clinic.

Diagnosis Tooth Chart

Click **Menu**. The patient menu's tooth chart gives an overview of the open diagnosis.

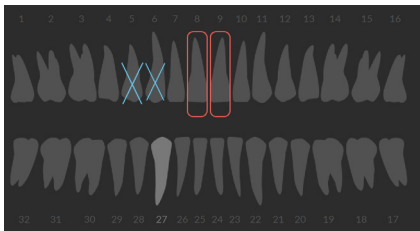





Note

If the patient is younger than eight years old, the tooth chart for primary teeth is shown. Please keep in mind to change the teeth manually to go to an adult tooth chart when the patient is growing up.









Edit the Tooth Chart



To edit the tooth chart, click a tooth on the tooth chart and select one of the following:

Icon	Action	Explanation
	Exchange	Exchange a primary tooth with an adult tooth. This option is available if the child tooth has a corresponding adult tooth. If the tooth is exchanged, all the findings of the primary tooth are deleted and the adult tooth is set to healthy. Note Child dentition is displayed for patients younger than eight years old.
X	Not present with gap	This tooth is missing and there is a gap on this location.
	Impacted	This tooth will be impacted (often used for wisdom teeth).
	Insert	Insert a tooth, e.g., adult molars in a child dentition.
	Not present without gap	Indicate hypodontia.

Diagnosis Data

Below the tooth chart, the scans and the images for the open diagnosis are shown per data type, and sorted by acquisition date.

-  3D X-ray
-  OPG (panoramic image)
-  Intraoral images
-  Cephalogram
-  Clinical pictures
-  Screenshots
-  Face scan
-  IO scan

- Click another 3D X-ray to switch between 3D X-rays.
- To start acquiring data directly in DTX Studio Clinic and add it to the open diagnosis, click **Capture** . Alternatively, click  on the menu bar.

Set the 3D and IO scan Viewer Background

To change the default background color of the 3D or IO scan viewer:

1. In the **Preferences** window, click **3D viewer**.
2. Select **Solid color**.
3. Select a color in the drop-down menu, or select **Custom** to select any other color.
4. Click **OK**.





Set the Default Image Zoom Level

To set the default zoom level of shown images:

1. In the **Preferences** window, click **Workspace**.
2. In the **Default image size** list, select the default magnification value.
3. Click **OK**.

Interact with the Mouse

Use the mouse to interact with the viewers.







	Action	3D Viewer	Other Viewer Types
	Right-click and drag	Rotate the 3D model	Brightness/Contrast (default) or zoom in and out
	Ctrl + click and drag, or Cmd + click and drag	Pan	Pan
	Shift + click and drag	Zoom in and out	Zoom in and out
	Scroll the wheel button	Zoom in and out	In reslice viewer only: scroll through slices




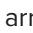


Workspaces




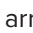







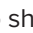
Select a workspace from the workspace bar or use the corresponding shortcut key, if available.

Note

Only the workspaces for which images or data have been added to the diagnosis are shown.

Workspace	Description	Shortcut key
3D patient	<p>To inspect the loaded model from all sides, use the mouse actions (see page 32) and the keyboard shortcuts. Alternatively, use the standard clinical view icons:</p> <ul style="list-style-type: none">  Frontal  Posterior  Left lateral  Right lateral  Cranial  Caudal <p>Press F2 again to go to the IO scan workspace (if available).</p>	F2
3D panoramic	<p>The 3D panoramic radiograph is generated based on the loaded 3D X-ray.</p>	F3
Tooth	<p>Navigate to a specific tooth and compare all 2D and 3D data using the SmartPanel™ Layout tab. Annotate the selected tooth (see page 37).</p> <p>The vertical slider on the perpendicular viewer rotates the slices around the tooth rotation axis. Depending on the situation, the following slice hints indicate the orientation of the reslice:</p> <ul style="list-style-type: none"> – Oral/Buccal (O / B) – Mesial/Distal (M / D) – Left/Right (L / R) <p>To adjust the rotation axis, see page 38.</p> <p>Press F4 again to go to the endo workspace (if available).</p>	F4


Workspace	Description	Shortcut key
Endo	<p>Focus on a specific tooth for endodontic diagnostics and procedures. To display the tooth pulp, click the Endo 3D visualization on the SmartPanel™ Viewer tab.</p> <p>Note</p> <p>This workspace is available if a 3D X-ray is loaded and when tooth annotations are defined.</p> <ul style="list-style-type: none"> – The 3D viewer is focused on a tooth of interest. – The Tooth cross-sections viewer is a cross-sectional viewer that shows several horizontal tooth cross-sections. – Once the root morphology is defined (see page 39), the root canals are visualized. <p>Press F4 again to go to the tooth workspace (if available).</p>	F4
Implant	<p>Plan and inspect implants. This workspace consists by default of 3 viewers, the OPG viewer, the snapped to implant viewer and the cross-sectional viewer.</p> <p>Click and drag a cross-sectional reslice in the OPG viewer to navigate to a desired position.</p>	F9
3D inspection	Navigate to a specific point outside the tooth range to inspect it.	N/A
TMJ	Inspect the condyle heads and the temporomandibular joint areas.	N/A
IO scan	Inspect and compare IO scans. Press F2 again to go to the 3D patient workspace (if available).	F2
OPG	View a 2D panoramic radiograph (panorex) or multilayer pan images.	F5
Intraoral	<p>Inspect the intraoral images on a layout, for inspecting e.g., a Full Mouth X-ray series.</p> <ul style="list-style-type: none"> – Double-click an image to use additional workspace functionalities: image filters and SmartLayout (see page 36). – To return to the initial layout overview, double-click the image again or press Esc. – Switch to another image by clicking on a thumbnail in the overview on the SmartPanel™ Layout tab. Alternatively, use the     arrow keys. – Multiple images can be stacked in the same placeholder. Click  to show all the images and click  to compare them. 	F6

Workspace	Description	Shortcut key
Ceph	View the frontal and/or lateral cephalogram. Use the Generate the 3D Cephs tool to calculate cephalograms based on the loaded 3D X-ray, or import 2D cephalograms.	F7
Clinical pictures	View the clinical pictures for the patient. <ul style="list-style-type: none"> – Double-click an image to use additional workspace functionalities: image filters and SmartLayout (see page 36). – To return to the initial layout overview, double-click the image again or press Esc. – Switch to another image by clicking on a thumbnail in the overview on the SmartPanel™ Layout tab. Alternatively, use the     arrow keys. – Multiple images can be stacked in the same placeholder. Click  to show all the images and click  to compare them. 	F8
Intraoral pictures	Similar to Clinical pictures, but containing the intraoral camera pictures. When a tooth is selected on the tooth chart and the intraoral camera is used for image acquisition in the tooth workspace, the acquired images are automatically assigned to the selected tooth and the assigned tooth numbers are shown in the clinical pictures workspace. <ul style="list-style-type: none"> – Double-click an image to use additional workspace functionalities: image filters and SmartLayout (see page 36). – To return to the initial layout overview, double-click the image again or press Esc. – Switch to another image by clicking on a thumbnail in the overview on the SmartPanel™ Layout tab. Alternatively, use the     arrow keys. – Multiple images can be stacked in the same placeholder. Click  to show all the images and click  to compare them. 	N/A
Intraoral camera	Dedicated workspace for intraoral camera acquisition.	F10
Recent images	The Recent images workspace shows all recently imported or acquired images. By default, the workspace shows the images of the last seven days. To change this, go to the DTX Studio Clinic preferences.	F12

Customize Workspaces

1. In the [Preferences](#) window, click [Workspace](#).
2. In the [Default workspace](#) list, select the workspace to be shown by default when opening DTX Studio Clinic. The standard setting is [Most recent data](#), the workspace associated with the most recently acquired or imported image.
3. Optionally, change the number of days in the [Recent images](#) field for images to be displayed in the [Recent images](#) workspace. The default value is 7.
4. Click [OK](#).

Show all Related Tooth Information with SmartFocus™

To activate SmartFocus in a supported viewer, press the spacebar. Alternatively, click  on the top menu bar.

- Click on a tooth region to go to the tooth workspace and optionally load the data for the specific tooth in the viewers.
- Click on a region outside the tooth range to go to the 3D inspection workspace.


Customize Views with SmartLayout™

Customize a workspace by adding or removing viewers via the SmartPanel™ [Layout](#) tab and by changing the viewer proportions.

- To add another viewer to the workspace, click a tile on the SmartPanel™ [Layout](#) tab.
- Re-click the tile to remove the viewer from the workspace.
- To change the proportion of the viewers, drag one of the window splitters.
- To close a viewer, click on the top-left window title. Select [Close viewer](#). Alternatively, press [Q].
- To sort by modality, date or to have selected images appear first, click the [Sort by](#) drop-down menu and select either [Modality](#), [Date](#) or [Selected first](#).
- To save the workspace layout, click [...](#) next to the workspace title and select [Save workspace layout](#). This layout is set as default layout for new patient diagnoses. To reset the viewers, click [Reset workspace](#).

Align IO scans with SmartFusion™

To align an IO scan with the 3D X-ray in the 3D patient workspace:

1. Click [Align on 3D X-ray](#)  in the [IO scan](#) tools menu.
2. Select an IO scan and click [Next](#).
3. If needed, indicate corresponding points and use the [Bone threshold](#) slider to adjust the visualization.
4. Check the alignment.
5. Click [Finish](#).




Use the Thumbnail Bar

At the bottom of the intraoral workspace and the clinical pictures workspaces, a thumbnail bar contains the images that are added to the diagnosis but not shown in the workspace viewer.

- Drag an image from the thumbnail bar and drop the image on a placeholder.
- If the placeholder already contained an image, this image is replaced by the new image and the old image is re-added to the thumbnail bar.

Add Diagnostic Findings

The SmartPanel™ **Findings** tab allows you to note dental pathologies, jaw problems or other diagnostic findings on tooth level.

- To add a predefined diagnostic finding to the tooth, click **Add finding**  in the **Diagnose** tools menu, or on the SmartPanel™ **Findings** tab. Select a finding. Optionally, include a screenshot by clicking **Screenshots**  on a finding.
- To remove the finding, hover over or select the finding, click  and select **Delete**.
- To add a custom diagnostic finding, enter the custom finding text in the search field and press Enter or click **Add**.
- Click the drop-down menu to assign a status, if needed.

Notes


In the tooth workspace, the status is visually indicated on the tooth chart as well.

If the finding is added in the tooth workspace, the finding is added to the specific tooth.

If the finding is created in another workspace, click the tooth number placeholder and type the tooth number to assign the finding to a specific tooth.

SmartPanel™ Tooth Chart


In the tooth workspace and endo workspace, a tooth chart is shown above the SmartPanel™ tab.

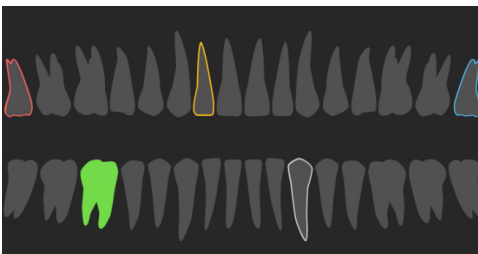

- The active tooth, for which the data are shown in the workspace, is highlighted in green.
- Select another tooth by clicking a tooth on the tooth chart.
- Below the tooth chart, click **<** to go to the previous tooth, or click **>** to go to the next tooth.
- A tooth with at least one finding has a colored outline. The color depends on the treatment status of the finding.

Treatment status	Color	Description
Condition	Gray	The finding is not critical, but may need to be watched to track its evolution over time.
Treatment plan	Red	The finding needs to be treated.
Follow-up	Orange	The finding is discovered at an early stage and should be watched.
Completed	Blue	The treatment for this finding has been completed.

Adjust the Tooth Rotation Axis in the Tooth Workspace

1. Click [Edit axis](#)  or press [A].
2. An orange cross is drawn on top of the perpendicular and the parallel viewer.
3. Drag the mouse inside the viewer to rotate the image data around the center point of the viewer.
4. Press [A] again or right-click to finish the action.

Clipping the 3D Volume

On the SmartPanel™ [Viewer](#) tab of the 3D patient, Tooth and 3D Inspection workspace, select [Clip 3D Visualization](#), to hide a part of the 3D volume and inspect certain areas of the volume.

Use Dental Intraoral Cameras in Workspaces

Acquire intraoral images with a USB dental intraoral camera directly in the intraoral camera workspace or from within another workspace:

1. Click the [Intraoral camera](#) workspace tab or in any workspace, click the SmartPanel™ [Cameras](#) tab.
2. Click a camera device tile, if needed.
3. Press the device button to acquire the image. Alternatively, click [Capture image](#) at the bottom.

Tools

The toolbar provides you with tools to diagnose, measure, plan treatments and edit the scan data.

Not all the tools will be available in all the workspaces. Unavailable tools are grayed out.



Warning

Measurement accuracy depends on the image data, the used scanner hardware, its calibration and acquisition settings. The measurement cannot be more precise than the resolution of the image. DTX Studio Clinic software reports the value, rounded to one digit after the decimal point, based on user-picked points.

Click one of the toolbar tabs to go to the underlying tools.

Diagnose tab



Set the slice thickness of a 3D reslice viewer. Click on the 3D reslice viewer and drag horizontally to set the X-ray thickness. Right-click to finish.

Note

To set a default slice thickness, go to the [Image settings](#) tab in the DTX Studio Home settings or DTX Studio Clinic preferences. In the top-right drop-down menu, select [3D Reslices](#), [Panoramic](#) or [Tooth](#). Select a preferred thickness in the [Slice thickness](#) drop-down menu.



Drag on a viewer to adjust the brightness and contrast:

- Horizontally: to change the contrast.
- Vertically: to change the brightness.

Note

When the brightness and contrast tool is used with grayscale images, the level and window values are updated accordingly.



Magnify a certain area of an image (default setting) or compare applied filters to the original image. Use the minus and plus keys (or Shift + Plus key when using macOS) to adjust the magnification level. To change the default settings, go to the DTX Studio Clinic preferences.



Inspect the underlying reslice when clicking on a 3D model.

- The reslice is shown in the overlay slice explorer window.
- Active image filters and the slice thickness are applied to the slice explorer view as well.
- Scroll to go through all the reslices.
- While inspecting the underlying reslice, the 3D model remains rotatable.



Capture a screenshot. It is added to the clinical pictures workspace, the SmartPanel™ [Layout](#) tab and the patient data. The captured screenshot can be added to a report ([see page 37](#)).



Add a finding to the SmartPanel™ [Findings](#) tab.



Analyze the airway. Indicate landmarks to create a box surrounding the region of interest. Click [Done](#). The airway volume and the most narrowing area is visualized in the 3D patient workspace.

Diagnose tab



Indicate a nerve canal. Click the first anchor point. Then click every next anchor point. Right-click to finish.

- Adjust the nerve canal annotation by moving the anchor points on the viewer.
 - All the anchor points will appear as one line on the SmartPanel™ **Annotation** tab.
-



Draw a custom reslice line in any reslice viewer of the 3D patient workspace (coronal/sagittal/axial) to create a custom reslice to inspect the (CB)CT data in detail. For example to mark and inspect the root canals and to make annotations.

- Move the custom reslice by clicking and dragging the reslice line.
 - Rotate the custom reslice by clicking and dragging one of the reslice endings.
-



Define the **root morphology** by indicating reference points in the apical section of each root canal.

1. Scroll through the slices to find an optimal position.

Note

If needed, re-indicate the tooth center in the **Horizontal** viewer by clicking and dragging the center.

2. Click a position in one of the viewers.
 3. The root canal visualization is immediately shown in the **Root canal** viewer.
 4. Click **Add point** to add a new reference point, if needed.
 5. Click **Done**.
-

Annotate tab



Add text to an image



Draw segmented lines with the pen. All the lines will appear as one annotation on the SmartPanel™ **Annotation** tab.



Draw free-form lines with the pencil.



Draw a circle.







Draw an arrow.





Select the line thickness for an annotation.

Measure tab


-
- | | |
|---|--|
|  | Measure the HU value of a point. Click a point in the scene to measure the HU or the gray value. |
|---|--|
-
- | | |
|---|---|
|  | Measure a linear distance. Click the two points between which you want to measure the distance. If the image has not yet been calibrated, enter a Reference value. The calibration measurement will be shown in the scene and the calibration object is added on the SmartPanel™ Annotation tab.


The measurement (and its accuracy) is shown. |
|---|---|
-
- | | |
|---|--|
|  | Measure segments. Click the first point. Then click every next point. Right-click to finish. |
|---|--|
-
- | | |
|---|---------------------------------------|
|  | Measure an angle. Click three points. |
|---|---------------------------------------|
-




Plan tab

-
- | | |
|--|---|
|  | Place an implant. This tool can be used in any workspace that contains (CB)CT data. |
|--|---|
-
- | | |
|---|--------------------------------------|
|  | Manage which implants can be placed. |
|---|--------------------------------------|
-

3D X-ray tab


-
- | | |
|---|---|
|  | MagicDetect is an AI-powered algorithm used to automatically set up the (CB)CT data and mandibular nerve annotation. All automatically detected points can be manually adjusted.


3D X-rays containing automatically detected attributes are indicated by an 'Auto' label in the bottom-right corner. |
|---|---|
-
- | | |
|---|--|
|  | Edit the orientation of the patient model. The 3D patient model can be orientated in the preferred position by translating and rotating the model in the 3D viewers.


<ol style="list-style-type: none">1. Click the pan icon  or the rotation icon  or press the [Tab] key to switch between rotation and translation mode. The selected mode appears in green.2. Drag the model until it is correctly aligned with the reference lines.3. Click Done. |
|---|--|
-
- | | |
|---|--|
|  | Adjust the OPG curve. Indicate the points and teeth as requested. When the teeth are not clearly visible, scroll or use the gray slider at the right side to adjust the position of the axial reslice to a plane showing the tooth setup (approximately converging with the occlusal plane).


If needed, adjust the curve: <ul style="list-style-type: none">– Click and drag individual control points to adjust the shape of the curve.– Click on the curve to add a new control point.– Click and drag the surrounding area to move the complete curve. |
|---|--|
-

3D X-ray tab

 Define the TMJ area. Indicate the condyle head position as shown in the wizard. Click Done. The TMJ workspace is opened, to compare the left and the right condyle head position and examine the temporomandibular joint area.


 Adjust the tooth positions. On the tooth chart, select the tooth that you want to calibrate. Drag the tooth indication to its correct position on the axial reslice. Adjust the axis of the tooth on the perpendicular reslice.

 Adjust the bone threshold. Click and drag horizontally on a 3D viewer to adjust the bone threshold value.

 Clean up the patient model by cutting out redundant parts. Click a point in the scene to start drawing around the section that should be removed. Right-click to confirm.

Note


To restore the original patient model, click **...** next to **3D visualization** on the SmartPanel™ **Viewer** tab. Select **Reset 3D model**.


 Generate an OPG. The panoramic (reslice) view is added to the patient data as a 2D image. The generated image is opened in the OPG workspace.

 Generate 3D cephalograms based on the imported 3D X-ray.


IO scan tab

 Automatically orientate IO scans.

 Align or realign an IO scan on the 3D X-ray via SmartFusion™.

 Fill the holes* of all jaw scans and diagnostic scans currently visible in the IO scan workspace. Choose either to fill small or all holes. Click **Fill holes**. The added texture is indicated in blue.

* For Windows only.

 Compare IO scans to follow up on gingival recession, tooth wear and other differences. Select an IO scan to compare with the reference scan. Click **Finish**.

By default, a colored distance map is applied. On the SmartPanel™ **Viewer** tab, select **Overlay** to show the two scans aligned with each other. Disable the comparison by turning the **Scan comparison** switch off.

 Calculate virtual teeth with SmartSetup.

Reports

Create Reports

To create a report containing findings or as template for patient-related letters:

1. In DTX Studio Clinic, open the patient menu.
2. Click [Export report](#).
3. Select a report template.
4. Click [Export report](#).
5. The report is exported in an editable .odt format and opened in the default text editor, e.g., Microsoft Office, LibreOffice, OpenOffice Writer.
6. Make any changes, if needed.
7. Save the report.

Add Custom Practice Logos

By default, the DTX Studio Clinic icon is added to a report header. To add a custom logo:

1. In the DTX Studio Home [settings](#) sidebar, click [Reports](#).
2. Click [Browse](#).
3. Select a new logo.
4. Click [OK](#).

Opening DTX Studio Implant

Connect DTX Studio Clinic and DTX Studio Implant


1. In the DTX Studio Home [settings](#) sidebar, click [DTX Studio Implant](#).
2. Click [Browse](#) to go to the location on the computer where DTX Studio Implant is installed.

Note

Set the location for patient data in case you need to add patient data to the patient record in DTX Studio Implant manually; that is if a patient record already exists in DTX Studio Implant or if intraoral scans are exported to DTX Studio Implant but are not aligned with the 3D X-ray.






3. Click [OK](#).

Launch DTX Studio Implant

1. Select the patient record in the patients list.
Note
At least one 3D X-ray must be available for this patient.
2. Click **Implant** .
3. Select **Open existing patient** or **Export to new patient**.
4. If there is more than one 3D X-ray, select the appropriate tile.
5. Click **Export**.
6. A success message appears. Click **OK**.
7. The patient record is created and/or opened in DTX Studio Implant.

Orders

Order a Surgery Plan, Surgical Template or Restoration

1. Select the patient record in the patients list.
2. Click **Order** .
3. Select **Surgery plan**  / **Surgical template**  / **Restoration** .
4. Select the patient data to send to the lab or the clinician.
5. Click **Continue**.
6. A draft order is created on DTX Studio Go. Add the missing data and send the order to the connected lab or clinician.
7. Click **Orders**  in the sidebar to view all your orders.

Note

Please note that some products described in the Instructions for Use may not be regulatory cleared, released or licensed for sale in all markets.

Focus Area Detection

What is Focus Area Detection?

DTX Studio Clinic allows to automatically detect focus areas on 2D intraoral X-ray images (IOR). The IOR capturing devices can either be digital sensors or analogue PSP plates.


Focus Area Detection is an AI-powered algorithm (artificial intelligence) using a convolutional neural network for image segmentation to locate regions of interest where a dental finding or capturing artifact might exist. An approved focus area is automatically converted into a diagnostic finding for that patient.

Supported focus areas for dental findings are caries, apical lesion, root canal defect, marginal defect, bone loss and calculus.

Supported focus areas for capturing artifacts are overlap and scratches.

Using Focus Area Detection

When intraoral images are captured or imported, Focus Area Detection is run automatically, to check whether the images contain areas that need special attention. You can see this by the blue line running over the images.


If an image shows potential dental findings, a blue **Focus Area Detection** icon  is shown in the top-left corner of the image combined with a number, indicating the amount of dental findings.

When an image does not contain the blue icon, either no possible dental findings have been detected or the image has not been checked. This does not mean that there are no potential dental findings. Caution should be exercised when using this feature.




Focus Area Detection

Click an image to open it. Focus areas are visualized on the intraoral images using focus area annotations. These annotations show the tooth number (if known), and the type of potential dental finding detected.

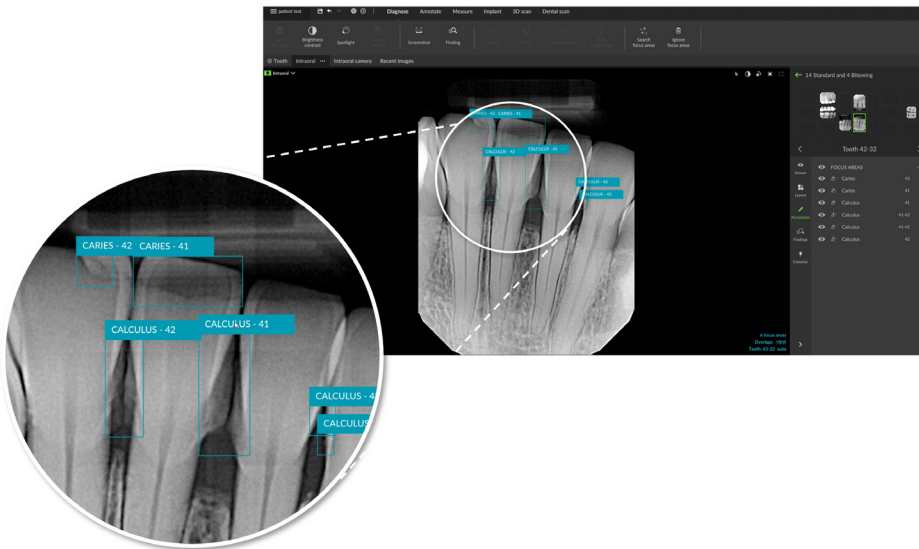

An indication of the whereabouts of the potential dental findings is visualized when hovering over the focus area. If you select the focus area by clicking it, it turns green and allows you to accept or to remove the focus area after your analysis.

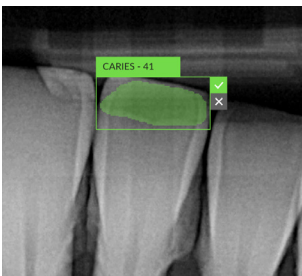


If you choose to accept it, the focus area is turned into a diagnostic finding automatically and will be added to the SmartPanel™ **Findings** tab.

The focus areas are listed on the SmartPanel™ **Annotation** tab as well and can be shown or hidden using the visibility icons of the SmartPanel™ 's visibility functionality.


Focus Area Detection

Overlaps and scratches

Focus Area Detection also checks for possible overlap of multiple teeth and/or scratches. If overlaps or scratches are detected, a notification is displayed.

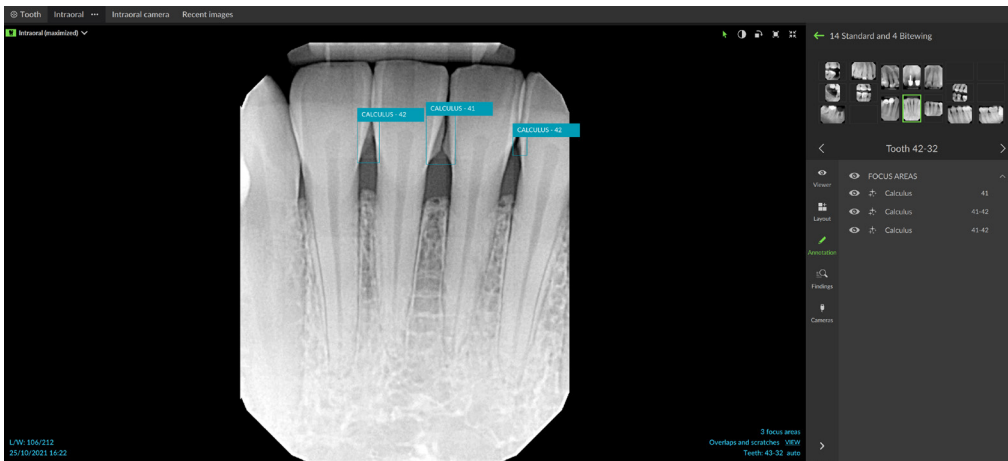

Click **View** to check the notification.

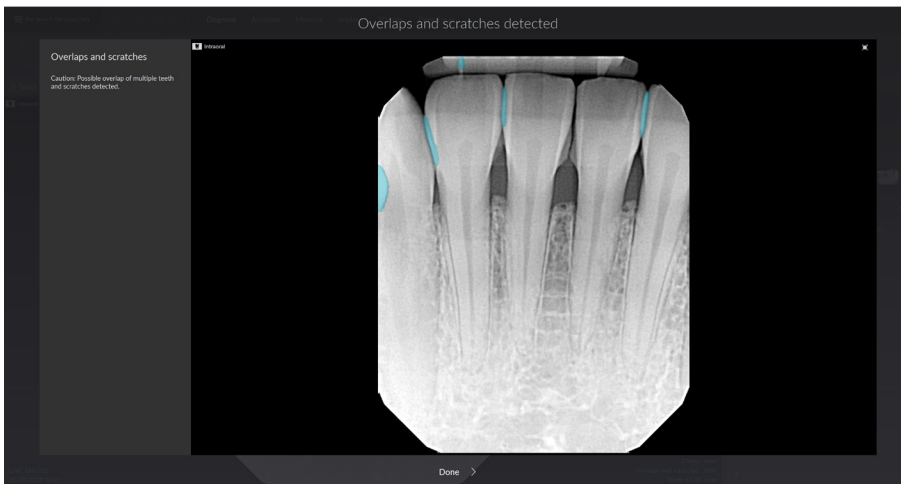



Nobel Biocare AB
Box 5190, 402 26
Västra Hamngatan 1,
411 17 Göteborg,
Sweden

www.nobelbiocare.com

Distributed in Australia by:


Nobel Biocare Australia Pty Ltd
Level 4/7 Eden Park Drive
Macquarie Park, NSW 2114
Australia

Phone: +61 1800 804 597



CH responsible person:

Nobel Biocare Services AG
Balz Zimmermann-Strasse 7
8302 Kloten
Switzerland

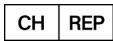

Distributed in New Zealand by:

Nobel Biocare New Zealand Ltd
33 Spartan Road
Takanini, Auckland, 2105
New Zealand

Phone: +64 0800 441 657

Distributed in Turkey by:

EOT Dental
Sağlık Ürünleri ve Dış Ticaret A.Ş
Nişpetiye Mah. Aytar Cad.
Metro İş Merkezi No: 10/7
Beşiktaş İSTANBUL
Phone: +90 2123614901



ifu.dtxstudio.com/symbolglossary
ifu.dtxstudio.com